




A Smarter
Ablation in a
Fraction of
the Time¹

QDOT MICRO®
Ablation Catheter

What's QDOT MICRO?

The Next Generation Ablation Catheter

- Consistent Lesion Creation^{*2}
- Advanced Diagnostics
- Simplified Workflow^{*+3}

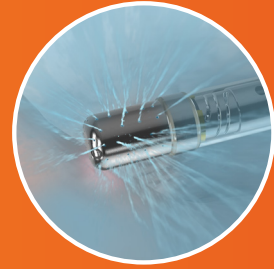


Optimized Irrigation

Irrigation rate varies automatically for optimized power delivery, ensuring the tip is within the allowed target temperature range.



Improved Temperature Monitoring

Temperature sensitivity that enables temperature control on an irrigated catheter with the addition of 6 thermocouples embedded into the tip.

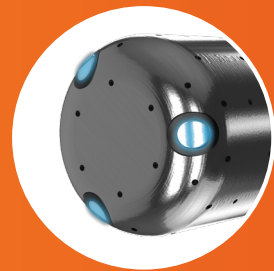

Higher Signal Resolution

QDOT MICRO[®] Catheter includes 3 microelectrodes providing high-resolution electrograms and discrete local signals.




Advanced Ablation

With QMODE+[™] temperature control, the generator delivers the high RF energy at the set power for the set short duration or until the target temperature is reached.

^{*}Based on pre-clinical data.

⁺When compared to THERMOCOOL SMARTTOUCH[®] Catheter and THERMOCOOL SMARTTOUCH[®] SF Catheter.

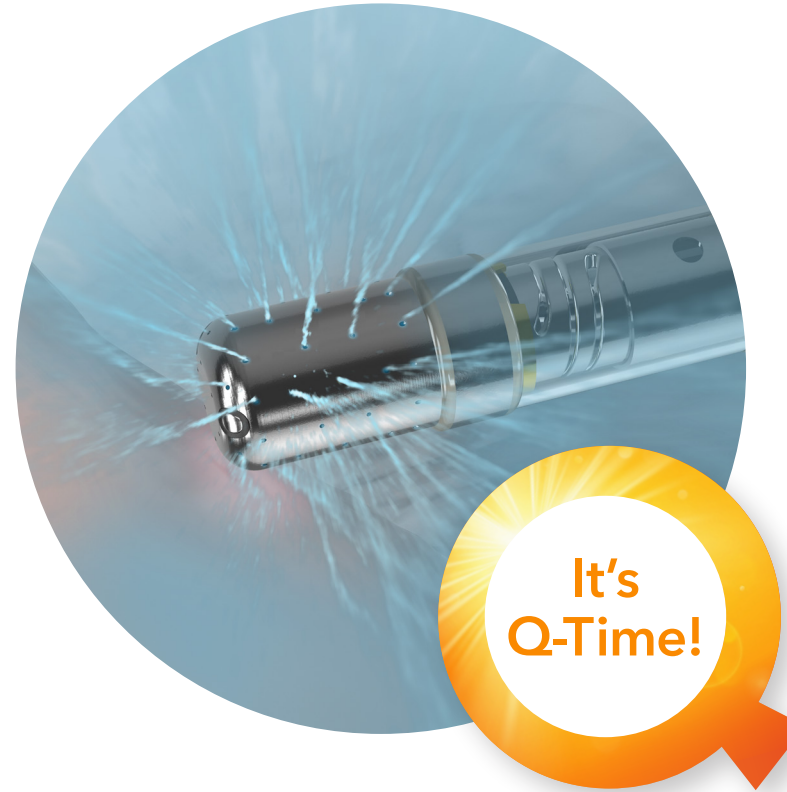

Optimized Irrigation Provides More Consistent Ablations


With QMODE[™], irrigation and power are automatically controlled using temperature feedback to maintain the tip at an allowed target temperature range while avoiding over-heating.**

QMODE[™] maximizes the power delivery by modulating the irrigation flow without exceeding the set target temperature.

Get in the Flow with Greater Control


- Reduced Irrigation Flow
- Improved Proximal Irrigation**
- Irrigation Varies Automatically to Ensure Tip Remains within Its Target Temperature Range.

**When compared to THERMOCOOL SMARTTOUCH[®] Catheter and THERMOCOOL SMARTTOUCH[®] SF Catheter.

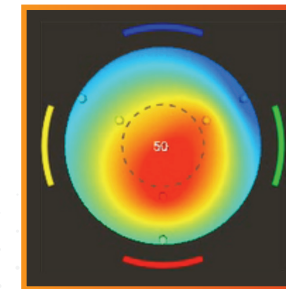

The Bullseye Interface

The bullseye tip display provides instantaneous feedback of ablation, in addition to stability.

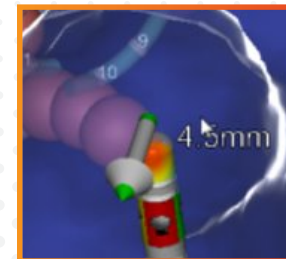

Intuitive Orientation Indicators

Force vector along with indication from tip display, as well as bullseye display visually confirms orientation.**


Stability You Can Count On

Seamless Integration with CARTO SMARTTOUCH™ contact force Technology***.


Thermocouple placement in close proximity to the tissue interface allowing a detailed temperature feedback system.

- Higher Temperature Sensitivity
- Enables Temperature Control on an Irrigated Catheter
- Real Time Catheter/Tissue Stability and Orientation Indication

Real-time Stability and Orientation

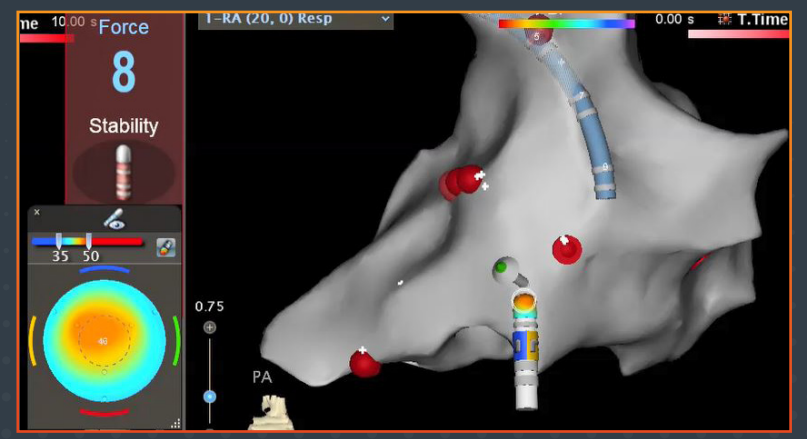

The QDOT MICRO® Unique Thermocouple Design

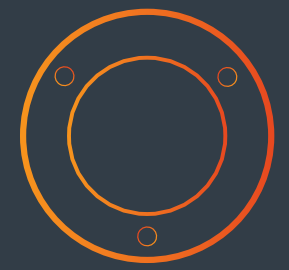


QDOT MICRO® Catheter, with its 6 thermocouples, improves temperature sensitivity that allows real-time temperature map display.




Thermocouples are in close proximity to tissue interface which allows for temperature feedback from tissue heating.

2D View for Different Tip Orientations During Ablation

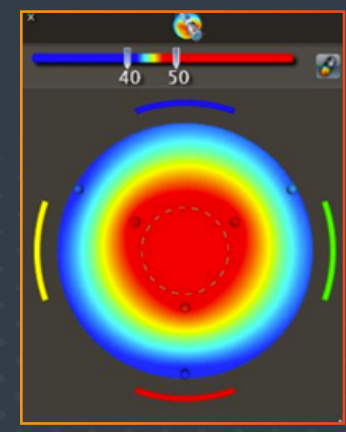

Perpendicular

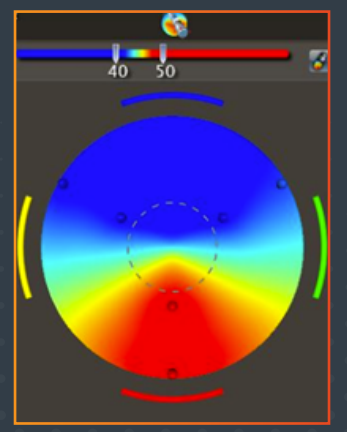


Parallel

The placement of the integrated thermocouples enabled improved temperature monitoring visualized through the tip and bullseye temperature displays.

*** Natale A, Reddy V, Monir G, et al. Paroxysmal AF catheter ablation with a contact force sensing catheter: results of the prospective, multi-center SMART-AF trial. J Am Coll Cardiol. 2014;64(7):647-656.

[In]formation You Can Count On

QDOT MICRO® enhances substrate mapping capabilities by delineating more accurately the border zone between scar and healthy tissue (smaller area), including identification of channels within the scar tissue.

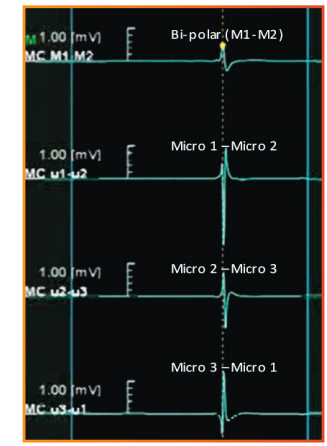


3 Microelectrodes Empower Information

QDOT MICRO® Catheter includes 3 microelectrodes providing high-resolution electrograms and discrete local signals.


Rich, detailed signal detection and enhanced user interface

- 3 Distal Microelectrodes 1.5 mm apart
- High-Resolution Electrogram Imaging and Discrete Signal Detection
- Effectively Characterizes Border Between Scar and Healthy Tissue

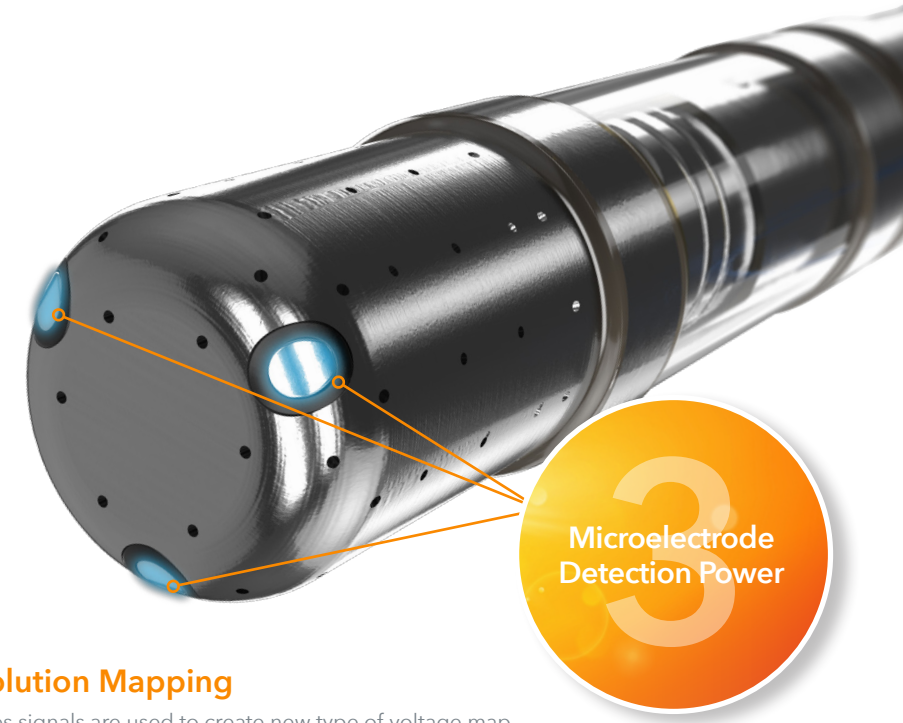

High Resolution Mapping

Microelectrodes signals are used to create new type of voltage map based on the bipolar voltages of the microelectrodes.

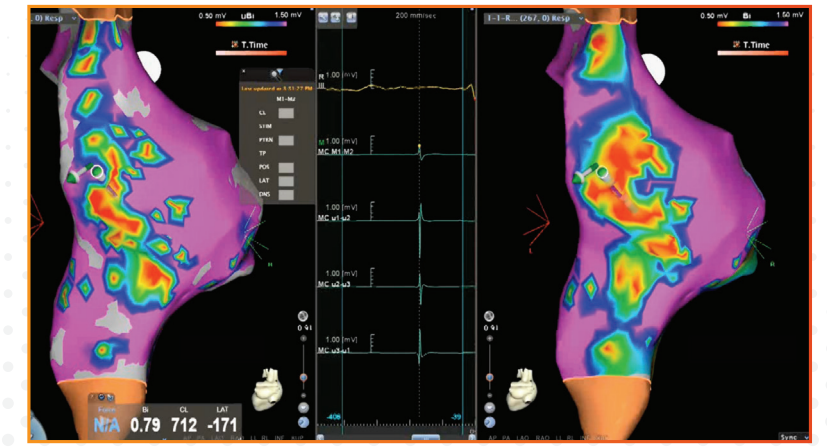

Microelectrode Map

Bi-polar Map (M1-M2)

Smarter Ablations In a Fraction of the Time¹

QMODE+™ with a temperature limit, resulted in wider ablation lesions, lesion depth similar to maximal depth achieved with standard ablation, and an overall improved lesion-to-lesion consistency.

90 Watts 4 Seconds


It's the Only Time
You'll Need.



High Power,
Short Duration
Energy Delivery
You Can
Count On

Resistive Heating vs. Conductive

RF lesion formation results from two thermal heating phases; resistive and conductive heating.

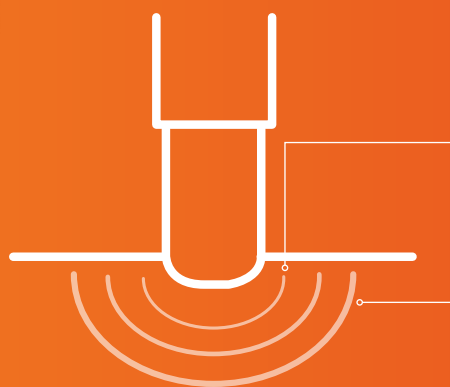

Resistive heating of tissue occurs near the contact point.

Conductive heating exchange into tissue occurs away from heat source.

QDOT MICRO® Delivers Smarter Ablations in a Fraction of the Time!

Ordering Information

Bi-directional with curve visualization

Ordering #	Curve type	French size	Electrode tip (mm)	Length (cm)
D139501	DD	8	3.5	115
D139502	FF	8	3.5	115
D139503	JJ	8	3.5	115
D139504	FJ	8	3.5	115
D139505	DF	8	3.5	115

Uni-directional with curve visualization

Ordering #	Curve type	French size	Electrode tip (mm)	Length (cm)
D139401	D	8	3.5	115
D139402	F	8	3.5	115
D139403	J	8	3.5	115



Biosense Webster, Inc.

33 Technology Drive
Irvine, CA 92618, USA

Tel: 909-839-8500

Tel: 800-729-9010

Fax: 909-468-2905

www.biosensewebster.com

Important information: Prior to use, refer to the Instructions for Use supplied with this device for indications, contraindications, side effects, warnings and precautions. For healthcare professionals only.

This publication is not intended for distribution outside of the EMEA region.

© Johnson & Johnson Medical NV/SA 2019 107915-200805 EMEA
(Item no 196960-211125 ME)

COPY-22029-C4

EC Representative | Biosense Webster

A Division of Johnson & Johnson Medical NV/SA
Leonardo da Vincilaan 15 | 1831 Diegem, Belgium

Tel: +32-2-7463-401 | Fax: +32-2-7463-403



Biosense Webster®

PART OF THE *Johnson & Johnson* FAMILY OF COMPANIES

1. Reddy VV, Grimaldi M, De Potter T, Vijgen JM, Bulava A, Duytschaever MF, Martinek M, Natale A, Knecht S, Neuzil P, Pürerfellner H. Pulmonary Vein Isolation with Very High Power-Short Duration Temperature-Controlled Lesions: The First-in-Human QDOT-FAST Multicenter Trial. JACC: Clinical Electrophysiology (2019). doi: <https://doi.org/10.1016/j.jacep.2019.04.009>.
2. Leshem E. 2018. High-Power and Short-Duration Ablation for Pulmonary Vein Isolation, Biophysical Characterization. JACC Clin Electrophysiol. 2018 Apr;4(4):467-479.
3. Design Verification Test Report for Evaluation of Qdot Micro-Electrode Catheter with Q-mode Ablation Mode: Beating Heart Animal Study, TR-0023068.